**Orthodontic Instrument  
INSTRUCTIONS FOR USE****Instrument Sterilization**

Dental instrument—for intended use only by dental professionals. Inspect, clean and sterilize before each use. Steam sterilize for at least 5 minutes at 273°F/134°C or 20 minutes at 250°F/121°C. Do not heat above 350°F/177°C. Use recommended chemicals only. Do not disinfect with phenols. Hinged instruments must be cleaned and sterilized in the open position. Always lubricate prior to sterilization.

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**Triumph  
Orthodontic Instrument  
Warranty****INSTRUMENT WARRANTY POLICY**

*Triumph* Orthodontic Instruments provide warranty against material defects and craftsmanship for the life of the product. Cutters are guaranteed for seven years, benders and utility pliers for ten years, and Mathieu pliers are guaranteed for one year. This covers corrosion, joint failure, and breakage under normal use. Proper cleaning and maintenance is required to maintain the warranty. This warranty may be void if instruments are not properly cared for, including cleaning and lubrication. *Ortho Organizers*® reserves the right to decide whether an instrument is to be repaired or replaced. Sharpening cutters and repairing minor tip damage is not covered by warranty as it is considered routine maintenance for normal use.

**Cleaning:**

Clean and remove any bulk debris immediately after use. Use an enzymatic cleaning solution in an ultrasonic cleaner to clean instruments. Your ultrasonic solution should be changed daily. Hinged instruments should be cleaned and sterilized in the open position. Discoloration and oxidation may occur if instruments are improperly cleaned. To increase the life of your instruments, clean them well. Use detergents and disinfectants suitable for use with medical and/or dental instruments. Follow manufacturer's instructions related to concentration and length of exposure.

**Lubrication:**

To increase the life of your instruments, routine lubrication is recommended. Prior to sterilization, lubricate instruments. This should be done at least weekly. Use only lubricants designed for precision hinged instruments. If using a dry heat sterilizer, be sure to use a lubricant that is compatible with the operating temperatures of your dry heat unit.

**Sharpening:**

Regular sharpening of your cutter will increase its service life, and its ability cut more effectively. Frequency of sharpening depends on: the frequency of use, as well as, the types and diameters of wires on which the cutter is being used. Check cutters frequently to assess if sharpening is needed.

**Sterilizing:**

Instruments should be in the open position when sterilized. The most efficient method is steam autoclave. *Triumph* Orthodontic Instruments are actually designed for use with steam autoclave. Steam autoclaves will not dull cutters. Other types of sterilizing methods include chemical vapor or dry heat. Using cold sterilants is not recommended. If used, they can chemically attack your instruments. All sterilizing units must be clean and operate properly.

**Corrosion:**

*Triumph* Orthodontic Instruments are very resistant to corrosion. You can prevent corrosion problems by properly following all manufacturers' recommendations. Discoloration may appear that may not be corrosion. Organic materials that remain can give the appearance of corrosion and be mistaken for rust. Never mix a corroded instrument with new instruments as oxidation may be spread to new instruments. It is recommended that a corroded instrument be discarded and replaced.

**Warranty Returns:**

Instruments which are defective under the terms of the warranty policy, and have not been misused or tampered with, may be returned to *Ortho Organizers* for repair or replacement at *Ortho Organizers*' option. Contact *Ortho Organizers* and obtain a Return Material Authorization (RMA) number prior to returning the pliers by calling Customer Service at 800.547.2000. Once an RMA number is assigned, ship the product with this RMA number visible on the outer shipping package, along with the original invoice or packing slip, to *Ortho Organizers, Inc.*, 1822 Aston Avenue, Carlsbad, CA 92008-7306. Please note that an RMA number is mandatory, and any product received by *Ortho Organizers* without an RMA number visible on the packaging will not be accepted.

Alternative maintenance providers may also be used. *Ortho Organizers* does not warrant actions by any maintenance service providers, and assumes no responsibility for the results of any maintenance services performed.

Ortho Organizers, Inc.  
1822 Aston Avenue  
Carlsbad, CA 92008  
USA  
Tel: 800.547.2000, +(1) 760 448 8600  
Fax: 800.888.7244, +(1) 760 448 8607  
USASales@OrthoOrganizers.com  
IntlOrders@OrthoOrganizers.com

Visit the *Ortho Organizers*' Web site at [OrthoOrganizers.com](http://OrthoOrganizers.com)

**mdi Europa GmbH**  
Langenhagener Str. 71  
30855 Langenhagen, Germany  
49 511 39 08 9530

OMA Member  
Orthodontic Manufacturers  
Association

INS126A  
Rev B